
RIDDER | *Assistent*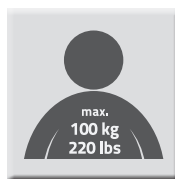

**MONTAGEANLEITUNG
USER MANUAL**

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Gebrauchsanleitung Rollator A0300301 / A0300317

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
User manual for walker A0300301 / A0300317


Dear customer,
thank you for choosing our product. Before use, please read the safety instructions carefully and keep the manual for further use. The manufacturer assumes no liability for incorrect assembly or improper use of the walker.

Delivery content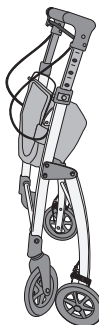

1x pre-mounted walker incl. basket (max. capacity basket: 12 L,
max. load limit: 5 kg / 11 lbs)

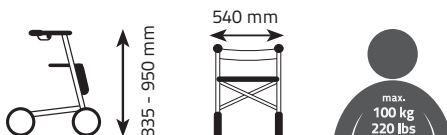1 x tray (max. load limit: 5 kg / 11 lbs)

1 x mounting and user manual

Technical Data:

Item no: A0300301 / A0300317

Model: Walker PIT

Max. dimension of the mounted walker (w x h x d): approx. 540 x 950 x 750 mm
(approx. 21.26 x 37.4 x 29.53 inch)

Smallest dimension of the folded walker (demounted front wheels and accessories):
approx. 540 x 970 x 280 mm (approx. 21.26 x 38.19 x 11.02 inch)

Weight of the walker (without accessories): approx. 6,6 kg (14.55 lbs)


Max. user weight: 100 kg (220 lbs)

Max. load limit basket/tray: 5 kg (11 lbs)

Grip height: approx. 835 - 950 mm (approx. 32.87 - 37.4 inch)

i NOTE!

Please open the packaging carefully, so that the product does not get damaged. Check the full delivery content for completeness and intactness. If you find any defects, please contact the service address given.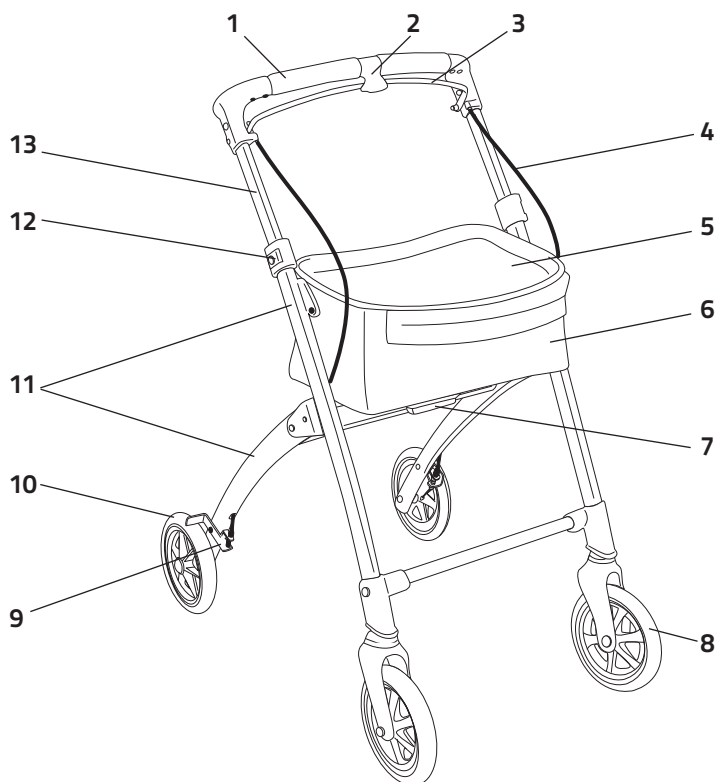

- | | |
|---------------------------------|---------------------------------|
| 1. Anti-slip handgrip | 8. Swivel wheel with reflectors |
| 2. Parking brake for handbrake | 9. Adjusting the brakes |
| 3. Hand brake | 10. Rear wheel with brake |
| 4. Brake cable | 11. Frame |
| 5. Tray | 12. Lock for height adjustment |
| 6. Basket | 13. Handle tubes |
| 7. Button for folding mechanism | |

i* NOTE!*Stability notes:**

The walker supports an individual weight up to 100 kg (220lbs), INCLUDING the loading of the basket and the tray. Tray and basket have a weight limit of 5 kg (11lbs).

 WARNING!	Warning, danger! Failure of respect the danger can result in serious injury or death.
 CAUTION!	Caution, danger! Failure of respect the danger can result in minor or moderate injury.
 NOTE!	Mind possible property damages

	Min. and max. height of the walker
	Max. width of the walker
	Max. user weight
	Can be used indoors
	Follow the instructions!
	Contact details distributor
	Date of manufacture
	Declaration of conformity (CE conformity)
	Medical product
	Mind hand injuries
	Disposal of the packaging

i NOTE!

Failure to observe these safety instructions can lead to personal damage. Please do not assemble or use the item until you have carefully read and understood the instructions for use. If you do not understand the instructions and information, contact a specialist or contact the dealer before using the article - otherwise damage could result. You should always consult your doctor or therapist for correct setting and use.

Intended Use

This walker serves as a walking aid for people with restricted mobility indoors. It can also be used as a means of transporting objects. The rollator is only intended for private purposes and is not intended for use in the commercial sector. Use the article only as described in the instructions for use.

The recommended service life of the product is a maximum of 5 years. This information assumes that the intended handling, care and safety instructions are observed.

Safety instructions** WARNING!**

- This walker is intended for indoor use only. Don't use it outdoors.
- The product is not for use by persons with limited physical, sensory or mental abilities designed. It may not be used due to a lack of knowledge or experience, unless instruction has taken place in the product, which has clarified the risks and the persons concerned are supervised by a person responsible for supervision.
- This walker is not a toy! Do not leave children unattended.
- Do not use the rollator under the influence of alcohol, medication or anything else Agents that affect your physical and mental cognition and motor skills.
- All tires must be in constant contact with the ground during use.
This guarantees the correct alignment of the walker. Therefore, do not use the walker on uneven surfaces such as sand or gravel. Also, pay special attention to bumps in the floor. If possible, avoid driving over them, as the wheels could get stuck in the depressions.
- Do not use the walker on very wet surfaces. Slippery surfaces strongly impair the slip resistance of the wheels.
- Do not use the walker when visibility is poor (e.g. in the dark).
- Do not use the device on stairs or escalators.
- Always transport objects in the basket or on the tray provided.
Pay attention to the max. load weight and the max. capacity.
- When adjusting the handle height, make sure that you are in an upright posture that is comfortable for you when using the walker. In order to prevent bad posture, let yourself be instructed by a professional for medical and rehabilitation products.

- Check that the handles are set at the same height on both sides and that the locking buttons are correctly engaged.
- Before each use, check the function of the brakes and make sure that the rollator is completely unfolded.
- Check the walker regularly for damage and wear. Have defective components replaced immediately and do not use the walker until it has been completely repaired. Do not make any changes to the product.
- Do not use the product if it is damaged!
- Always observe the maximum load limits of the rollator and the accessories.
Never exceed this limit, otherwise the function and stability of the product cannot be fully guaranteed.
- Do not stand or sit on the rollator and never use it as a stand-up or ascent aid!
- Since the walker is equipped with a folding function, there is always, and especially when unfolding and folding, the risk of being squeezed or pinched.
Always be careful not to pinch your fingers or hand.
- Risk of burns! Do not expose the walker to direct sunlight or other extreme heat sources, as the material can heat up quickly.
- Keep packaging material away from children.

Commissioning


CAUTION!


Risk of injury from trapping during the entire assembly!

The walker is delivered folded.

Unfold the walker: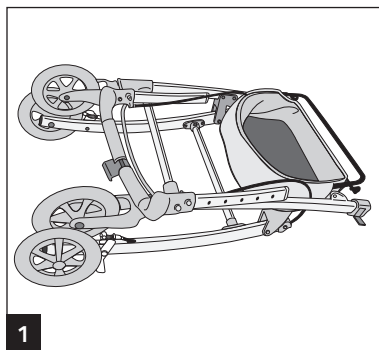



1

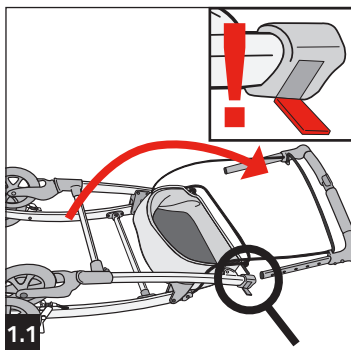1.1

1 Take the walker out of the packaging.

1.1 Put the handle tubes in the correct position. Make sure that the locking buttons on the sides of the frame are open.

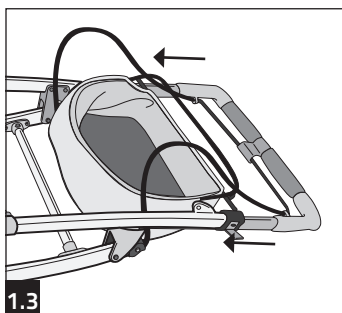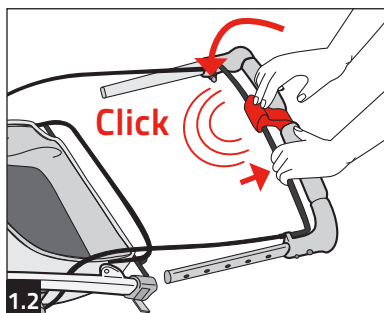1.2 Pull the brake lever towards the handle and enclose it with the locking.

1.3 Now insert the handle tubes into the tubes of the frame.

1.4 Finally, pull the front and rear wheels apart as far as possible until the frame clicks into place. A click is heard.

Adjust the handle height:

i NOTE!

The handle height of the walker can be adjusted in 5 levels using the lock. Assume an upright but comfortable position with your arms hanging loosely. In this position, your hands should be able to grip the handle of the walker **1**.


⚠ WARNING!



Incorrect handle height adjustment can lead to poor posture and other injuries **2** + **3**. Only use the walker with the handle height that is correctly set for you.

If you have not already done so, open the lock **4**. Adjust the handle on both sides to a position that is comfortable for you. Make sure that both handle tubes are at the same height. There are numbers on the tube next to the height adjustment for orientation **5**. Now close the locking buttons on both sides so that they snap into the designated hole on the handlebar. The locking button must click into place at the same number on both sides. Please note that the locking button must always be properly engaged in one of the 5 holes provided for it **6**.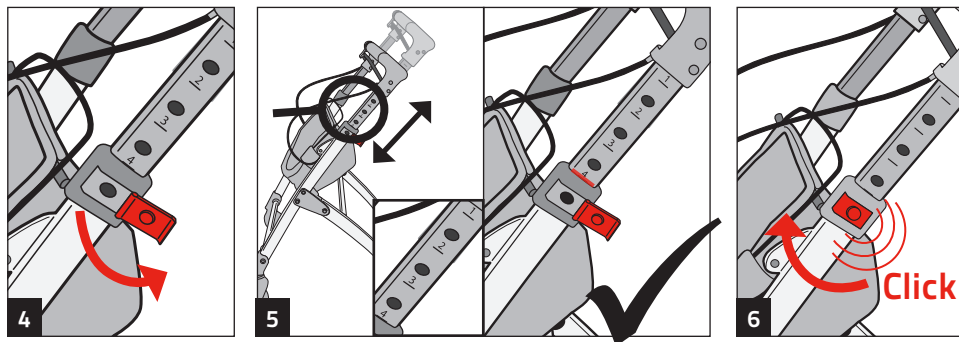

Unfold the basket:

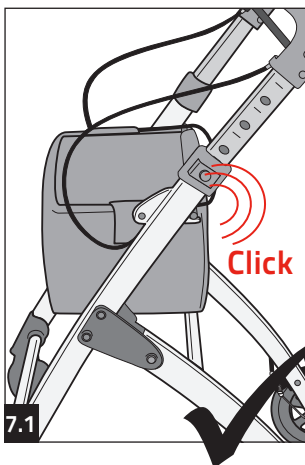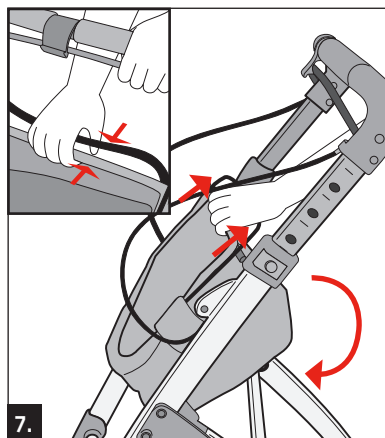
7. Loosen the basket by pulling back on the rear strut that holds the basket. The basket releases and can then be folded down.

7.1 As soon as a click can be heard, the basket is locked and can be used.

The basket can be removed from the walker at any time using the Velcro fasteners.

Release the parking brake: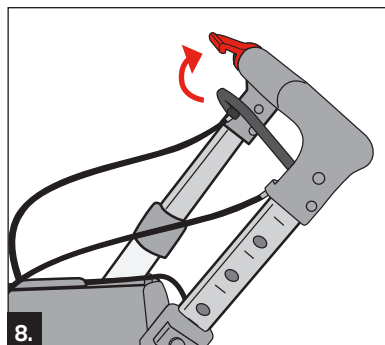

8. Now open the brakes by releasing the parking brake from the handbrake.

Your walker is now ready for use.

If necessary, place the tray in the holder.


Correct use in everyday life:

WARNING!

To achieve the best possible support, hold the walker close to your body. Only then is a safe walk guaranteed. If you take the walker too far away from your body, it can roll away easily and you may not be adequately supported.


Activation of the brakes: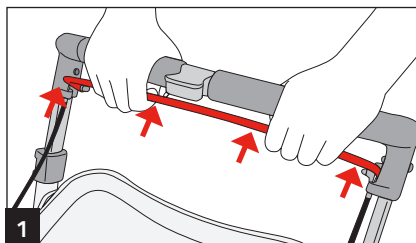

1 Handbrake (while walking)



Pull the brake lever up towards the handlebar with both hands at the same time and with the same intensity. To loosen it, just let it loose again. The harder you pull, the greater the braking power. If the braking power is too low despite firm pulling, the brake may have to be readjusted.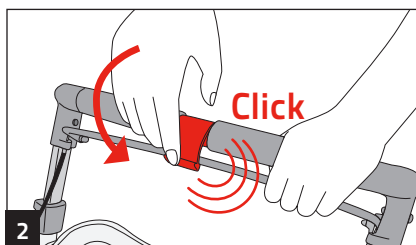

2 Lock the brake (when not in use)



Pull the handbrake towards the handle as described in point 1. Then clamp the brake lever with the aid of the parking brake so that the handbrake cannot be released again.

Fold: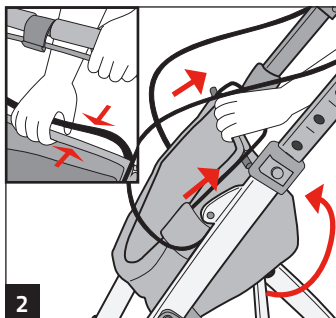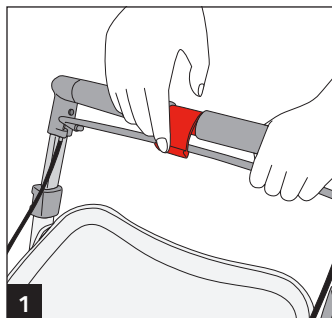

1 Lock the brake and remove the tray.

2 Fold the basket in by pulling the two struts together and folding the basket upwards.

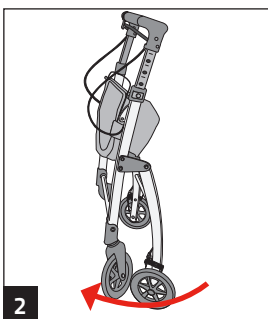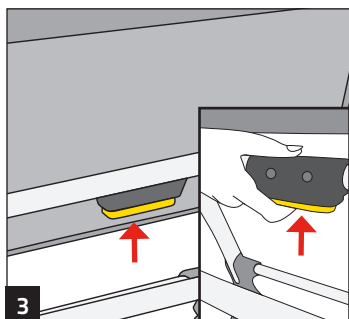3 There is a yellow button under the basket. Press this to unlock the walker and keep it pressed during the folding process.

2 Now fold the rear wheels of the walker as far as possible towards the front wheels. If necessary, adjust the handle height to the lowest level.

Storage:

For storage, the parking brake should always be activated.

Store the walker in a dry place by room temperature (approx. 20 ° C). Protect it from frost, direct sunlight or other weather influences. Keep the product out of the reach of children.

Cleaning and care:

Please note the cleaning and care instructions. Improper operation can damage the product.

Clean the walker gently with water and a standard commercial cleaner. Then dry the product with a cloth.

Do not use any aggressive cleaning agents and always make sure not to expose the walker to a hard water jet (e.g. from a high pressure cleaner), as this could damage small parts or the surface of the product.

Maintenance:

Before each use, check the functionality of the walker.

The following parts require your special attention:

- Brakes
- Wheels
- Crew connections
- Overall appearance of the product
- Locks

Disposal of the walker:

Please dispose the product in accordance with the applicable regulations via an approved waste disposal company or via the municipal waste facility. If necessary, inform yourself about correct and environmentally friendly disposal.

Disposal of the packaging:

Please dispose the packaging environmentally friendly. Make sure that you separate the waste properly.

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RIDDER GmbH